

Omnicaam FlexiGlass Lens protector install:

1. Please note that the Omnicaam is NOT a perfect square. The lens is a rectangle. Please make sure to double check you have it the correct way.



2. Once you have it the correct way. Use one of the Sticker pull tags included in the kit, to secure the Flexiglass in its holder.



3. Now peel the plastic off the back. Be careful not to take the Flexiglass out of the holder. (It should be secured by the sticker pull tag)

Once the plastic backing is off, pop out the Flexiglass from the holder using the sticker tag.



You can now install, simply line it up and apply even pressure with your finger. Remove the sticker pull tag and holder, While holder down the Flexiglass with your finger.

